# POWERGRIP

Bipolare Koagulationszange  
Bipolar Coagulation Forceps

# POWERGRIP

## Bipolare Koagulationszange Bipolar Coagulation Forceps

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### Quick exchange of electrode inserts

The different inserts can be exchanged in only a few seconds: The electrode is screwed into the shaft, inserted in the handle and locked in place by means of a click-stop. At the same time the connection to the integrated electric connector is established.

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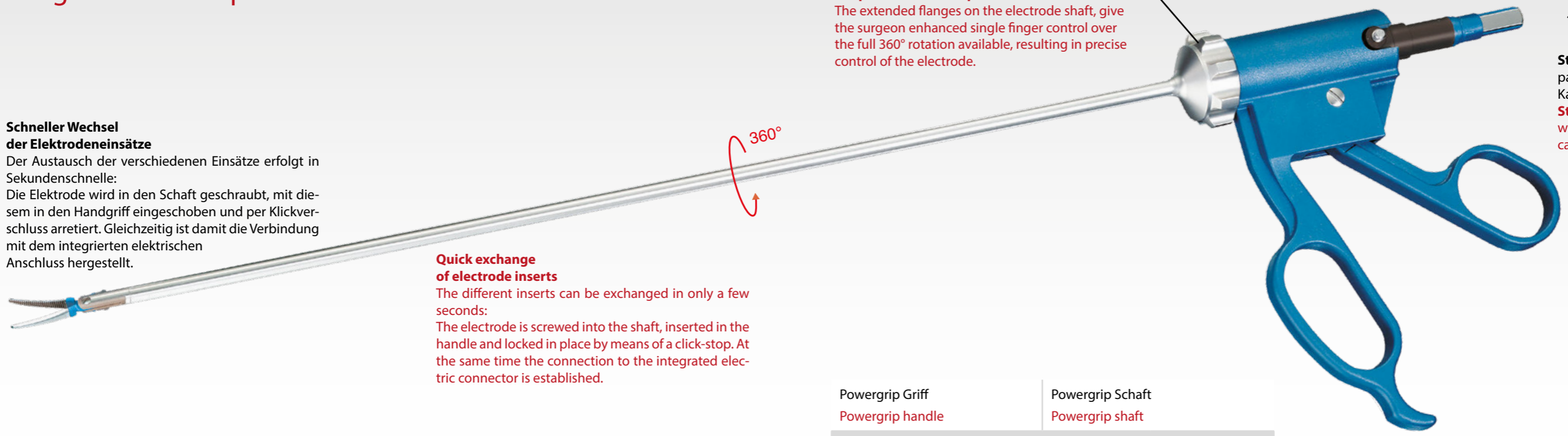
### Simple one handed operation

The extended flanges on the electrode shaft, give the surgeon enhanced single finger control over the full 360° rotation available, resulting in precise control of the electrode.

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Standard flat plug with common bipolar cable fitting

Powergrip Griff Powergrip handle	Powergrip Schaft Powergrip shaft	
Art. Nr. / Art. No. 82410001	Länge / length 34cm	Art. Nr. / Art. No. 82410034

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### Precise coagulation, more powerful grasping, better cutting and more exact work: the Bissinger POWERGRIP

The Bissinger POWERGRIP is an instrument used for grasping, cutting and bipolar coagulation in Minimally Invasive Surgery. It is characterized by a mechanism enabling well adjustable opening and closing of the jaws with very high pressure while grasping and cutting. This is accompanied by precise coagulation of the tissue. The jaws are moved by a double hinge which is connected electrically and mechanically to the handle by a slide rod. The connector for standardized bipolar cables is integrated in the particularly ergonomic handle made of high quality PEEK polymer. This saves the purchase of expensive special cables and prevents them from being connected incorrectly. We supply standardized bipolar cables for connection to practically all common high-frequency generators. The Bissinger POWERGRIP can be dismantled in three parts. This enables thorough and reliable cleaning and facilitates the execution of repair work or the replacement of defective parts. Beyond that, the Bissinger POWERGRIP offers a number of important further advantages:

- exchangeable electrode inserts for different applications
- easy handling
- excellent cleaning and ease of repair

	<b>Große Fasszange Large grasping forceps</b>	<b>82400010</b>
	<b>Pinzette gebogen Forceps, curved</b>	<b>82400011</b>
	<b>Fasszange gebogen Grasping forceps, curved</b>	<b>82400015</b>
	<b>Delta Fasszange Delta Grasper</b>	<b>82400014</b>

	<b>Dissektor Maryland gebogen Maryland dissector, curved</b>	<b>82400019</b>
	<b>Schere gebogen Scissors, curved</b>	<b>82400018</b>
	<b>Hakenzange Claw forceps</b>	<b>82400012</b>
	<b>Duck Bill Fasszange Duck Bill Grasper</b>	<b>82400008</b>

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**Bissinger – Focus on better solutions**

Bipolar and monopolar instruments  
 Accessories for electrosurgery  
 Surgical instruments  
 Design and development of instruments  
 to customer specifications  
 Service and repair

2-Bananenstecker* 	3 m	Art. No. 80100010
	5 m	Art. No. 80100013
U20 	4 m	Art. No. 80100260
Erbe 	3 m	Art. No. 80100017
	5 m	Art. No. 80100018
Martin, Berchtold Aesculap GK 55, GK 60 	3 m	Art. No. 80100019
	5 m	Art. No. 80100020
Valleylab, Lamidey, EMC 	3 m	Art. No. 80100021
	5 m	Art. No. 80100022

**CE 0297**

**Günter Bissinger**

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Hans-Theisen-Str.1

79331 Teningen/Germany

Tel. +49 7641 9 14 33 0

Fax +49 7641 9 14 33 33

eMail: [info@bissinger.com](mailto:info@bissinger.com)

Internet: <http://www.bissinger.com>

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\*Non-compliant with EU standards

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**Medizintechnik GmbH**